



Franke South Africa latest product:

DEKO190GT

DEKO Model 190GT Bedpan washer, 630 x 600 x 1460 mm, manufactured from Grade 304 stainless steel with an internal washing chamber no less than 480 x 360 x 640 mm. The Bedpan washer has the ability, but is not limited to, washing a maximum of 2 bedpans with lids and 4 urine bottles in a single cycle. The unit to be fitted with adjustable feet and to be freestanding with a front-loading door that prevents spillage. The outlet diameter must be 110mm and the unit is to comply with the ISO 15883-1 & 3 specification. The Bedpan washer must have a built-in tank for the recirculation of water with a pump of no less than 600l per minute to compensate for any drop in incoming water pressure. Unit to be manufactured with no less than 3 rotating and 7 fixed wash jets.

UNIQUE WASHING EFFICACY

Thanks to the unique, patented fresh water recirculation principle of the DEKO 190GT, the machine has a minimal need for energy and water in relation to the capacity and efficacy of the adjustable process cycles. Also the thermal disinfection of outlet, water tank and pipework in every cycle is of great importance and only available in the DEKO 190GT.

DEKO 190GT is intended for emptying, flushing, cleaning and thermal disinfection of reusable waste containers such as;

- supports for single-use bedpans
- hospital wash bowls
- urine bottles
- mop buckets
- suction bottles
- measuring jugs
- and all other reusable products used for similar purposes within patient care

POWERFUL PROCESS

Process results are guaranteed throughout:

- water pressure for washing is supplied by a high powered 600L/min recirculation pump.
- detergent is automatically dosed into the recirculated water by a pump.
- the unique working principle facilitates cleaning and steam disinfection of the entire system including wash chamber, water tank and outlet with minimal water and energy usage.
- disinfecting steam is generated by an integrated unpressurised boiler and run through the water tank, outlet and internal pipework wide selection of disinfection temperatures and cycle setups to secure the microbial inactivation impact.

Cycles can be altered to meet almost any special requirement, though the standard requirements are met with the fixed, tested and approved cycles pre-programmed by the manufacturer. The access code system protects any unauthorized changes to the set parameters. The washing spray patterns are delivered through 3 rotating + 7 fixed spray jets guaranteeing contact with all surfaces and crevices of items with varying design.

Contact Details:

Franke South Africa

Call Centre: 031 001 5000

Website: www.franke.co.za

Email: enquiry.fsa@franke.com

DEKO 190GT

DEKO 190GT | WARD WASHER-DISINFECTOR



FRANKE